Gebrauchsanweisung  
Instructions for use  
Mode d'emploi  
*for:*

Massage Form® Attach, Style 10475  
Massage Form® Shaper Attach, Style 11485  
Diamond Attach, Style 10487

## How to apply your Attach breast form

## How to remove your Attach breast form

## Daily Maintenance

1. Apply cleanser

2. Gently rub

3. Thoroughly rinse

4. Air dry in case

Ask your ABC retailer about the Cleanser , a specially formulated liquid cleanser that gently removes residue from the adhesive surface of the Massage Form® Attach, Style 10475 and Massage Form® Shaper Attach, Style 11485 and Diamond Attach, Style 10487

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Cleanser Ingredients / Inhaltsstoffe Cleanser: AQUA, ZEA MAYS (CORN) COB MEAL, SODIUM LAURETH SULFATE, COCONUT ACID, LAURETH-2, TRIDECETH-7, SODIUM C13-17 ALKANE SULFONATE, COCAMIDOPROPYL BETAINE, XANTHAN GUM, CELLULOSE GUM, ACRYLATES/C10-30 ALKYL ACRYLATE CROSSPOLYMER, PARFUM, PROPYLENE GLCOL, BENZYL ALCOHOL, METHYLISOTHIAZOLINONE, METHYLCHLOROISOTHIAZOLINONE CI 77891

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### 11. Entsorgung

Haumüll

**Thank you for choosing ABC Breast Care. ABC products are developed and manufactured with great care and knowledge of the needs of women who have undergone breast surgery.**

This instruction manual applies to:

- ABC Massage Form Attach, Style Nr. 10475
- ABC Massage Form Shaper Attach, Style Nr. 11485
- ABC Diamond Attach, Style Nr. 10487

Please note that not all ABC products are available in all countries.

#### **Scope of delivery:**

- Travel case for storage and travel
- 2 Covers for the adhesive surface (one of each: textile and plastic)
- Instruction manual

### **1. Indication for Use**

ABC Attach breast prostheses and partial breast prostheses are intended for external use only.

- ABC Attach breast prostheses and partial breast prostheses provide an anatomical and weight-based replacement of removed or missing breast tissue. This can be as a result of breast surgeries (such as mastectomy and/or breast conserving surgery), breast malformation, breast regression or natural breast asymmetry.

- ABC Attach breast prostheses and partial breast prostheses are worn directly on the skin and are intended to be worn with a bra. Wearing a bra provides additional security by holding the Attach breast prosthesis in place and ensures a safe and comfortable fit.
- The correct breast prosthesis and appropriate specialty bra are selected and fitted by trained professionals. Type, shape, size, weight and color of the breast prosthesis are chosen according to the anatomical conditions and needs of the wearer.
- ABC Attach breast prostheses and partial breast prostheses are intended for daily use and for wearing over longer periods of time.

### **2. Risks & Contraindications**

- There are no known risks or contraindications to be expected when wearing ABC Breast prostheses and partial breast prostheses under normal conditions and in accordance with the care instructions. No safety issues or performance impairments are expected during normal use. No restrictions on use are expected.
- We recommend wearing breast prostheses and partial breast prostheses only after the wound has fully healed and radiotherapy is complete. Wait at least 6 months after

the operation to begin wearing an Attach breast prosthesis.

- If you experience redness or rashes, please discontinue using the product and contact your doctor or medical device retailer.

### 3. Material

- All versions of ABC Massage Form® Attach are made of polyurethane film, silicone gel based on polydimethylsiloxane, color pigments and microspheres (in lightweight prostheses).
- The adhesive surface of ABC Attach breast prostheses are made of an adhesive silicone gel.
- All ABC breast prostheses are carefully controlled. Any discoloration of the breast prosthesis has no effect on the quality or wearing comfort.

### 4. Care Instructions

#### What to avoid:

- Avoid sharp objects. Be careful with sharp, pointed objects. Brooches, needles, fingernails, scissors, pet claws, and other sharp objects can cause permanent damage to your breast prostheses.
  - Avoid stretching, twisting, friction or excessive use of force when handling your breast prosthesis.
  - Avoid direct contact with perfume, hot surfaces or hot water.
- The breast prosthesis is resistant to sweat, moisture, water (both salt and chlorine water).

#### Proper skin care and skin preparation is essential for long-lasting adhesion:

- To achieve the best possible adhesion, please clean and dry your skin thoroughly in the adhesive area before applying the Attach breast prosthesis.
- Your skin should be free of soap residue, moisture, creams, powder and perfume. Skin care products containing oils may impair the adhesive surface of the Attach prosthesis.

#### Fitting the Attach Prostheses:

1. Stand in front of a mirror and put on your bra (as seen in the example on page 2).
2. Place your adhesive prosthesis (with the protective film still in place on the adhesive surface) in the bra cup (but not in the pocket). Position your Attach prosthesis in the desired location in bra cup.
3. Remove the protective film, preferably without touching the adhesive surface.
4. Press the bra cup with the adhesive side of the Attach prosthesis against your chest (as seen in the example on page 2).
5. Lastly, to ensure a smooth transition from chest to prostheses, make sure the edges of the breast prostheses adhere cleanly to the chest. To do this, hold the Attach prosthesis in place with one hand and gently press down the sides with the other hand.

### **Removing and Re-positioning the Attach Prostheses:**

- To remove the Attach prosthesis, gently loosen the prosthesis from the bottom upwards.
- To re-position the Attach prosthesis, remove it as described above and reattach it (as seen in the example on page 2).
- Please avoid touching the adhesive surface of the Attach prostheses. This may impair the adhesive surface of the Attach prosthesis.

### **How to use the textile covers:**

- The Attach breast prosthesis can be worn as a non-adhesive breast prosthesis in the bra pocket, by placing the enclosed textile cover on the adhesive surface.
- Please make sure to place the plastic side on the adhesive surface.

## **5. Cleaning Instructions**

- Daily cleaning of the adhesive surface is essential to maintaining a long-lasting adhesive strength of your Attach prosthesis.
- Ask your medical device retailer for ABC Cleanser for easy and thorough cleaning. Alternatively, you can use a mild, non-moisturizing liquid soap.

### **How to clean the Attach breast prosthesis:**

- Gently rub the cleanser or soap with your fingers circularly for 2-3 minutes over the entire adhesive surface.

- Thoroughly rinse the Attach prosthesis with lukewarm water to completely remove soap or cleanser residue.
- To dry: place your Attach prosthesis - with the adhesive side upwards - in the ABC travel case to allow the prosthesis to air dry (as seen in the example on page 3). Please do not dry the prosthesis with a towel. This avoids a build-up of lint on the adhesive surface, which may impair the adhesive surface of the Attach prosthesis.
- Should the adhesive have an excessive amount of residue, lint or dander covering the surface, we recommend using a soft brush in addition to the cleanser when cleaning the breast prostheses.

### **How to clean the textile covers:**

- Hand wash with mild soap, rinse in lukewarm water and let air dry.

## **6. Warranty**

- When worn under normal conditions and in compliance with the instructions for use, we guarantee a product service life of 2 years.
- Should your breast prosthesis become defective, please contact your medical device retailer. In the event of a production or material defect, ABC will replace the breast prosthesis should the defect happen within two (2) years from the date of purchase.

- The warranty does not include the strength or functionality of the adhesive area.
- The lifetime, functionality and strength of the adhesive surface varies from wearer to wearer. This depends on its skin condition, body hygiene and care of the prosthesis.

## 7. Incident Reporting

Only serious incidents relating to the product, which lead to a serious deterioration of the state of health or death, must be reported to the manufacturer or competent authority of the EU Member State. Serious incidents are described in Article 2(65) of Regulation (EU) 2017/745 (MDR).

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ABC Breast Care GmbH