ABC Nipples REF 907

Gebrauchsanweisung • Instructions for Use • Bruksanvisning  
ABC Bröstvårtor • Käyttöohjeet • Bruksanvisning •  
Brugsanvisning

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## 10. Entsorgung

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# **English: Instructions for Use**

Thank you for choosing ABC Breast Care. ABC products are developed and manufactured with great care and knowledge of the needs of women who have undergone breast surgery.

This instruction manual applies to ABC Nipples, Art./REF 907

Please note that not all ABC products are available in all countries.

## **Scope of delivery:**

- 1 pair of ABC Nipples in a box
- Instruction manual

You can also find the instructions for use online at:

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## **1. Intended Use/ Indications**

- ABC Nipples are intended for external use only.
- ABC Nipples are intended for external optical balancing of the nipples, after mastectomy and/or breast-conserving surgery (BET) or plastic surgery of the female breast.
- ABC Nipples are self-adhesive and are placed on the breast prosthesis or worn directly on the skin. A well-fitting special bra gives the wearer additional security.

- ABC Nipples are intended for daily use and for wearing over extended periods of time.

## **2. Risks/ Contraindications**

- Nipples are for external use only and must not come into contact with open wounds.
- We recommend that the self-adhesive nipples are only worn directly on the skin after the wound healing of the surgical scar and, if necessary, radiation therapy have been fully completed, at the earliest 6 months after the operation.
- If the nipples are used properly, no application risks or contraindications are expected or known.
- If you experience redness or rashes, please discontinue using the product and contact your doctor or medical device retailer

## **3. Content of Material**

- All versions ABC Nipples are made of polyurethane film, silicone gel based on polydimethylsiloxane and color pigments.
- 3 colors and 3 sizes are available.
- ABC nipples are carefully controlled for quality.

## **4. Application Instructions**

- ABC Nipples are resistant to sweat, moisture, water, seawater and chlorinated water.

- Avoid direct contact of nipples with hot surfaces or hot water
- There are no known contraindications when used properly.
- Please be careful with sharp objects (fingernails, scissors) which can damage the silicone nipples.

### **Application of a Nipple**

- Determine the desired position before application.
- Remove ABC Nipple from the foil and place it on the breast prosthesis or breast.
- ABC Nipples can be removed at any time and, thanks to their self-adhesive properties, can be removed and reapplied again and again.
- If you need to correct the position, carefully detach and reapply the nipple.

### **5. Care Instructions**

- Moisten ABC Nipples with water after wearing and clean thoroughly with neutral, non-regreasing soap. Then rinse with plenty of lukewarm water and allow to dry.
- Please do not towel dry to avoid accumulation of lint on the adhesive surface.
- Put the nipples back on the protective film after use/cleaning and place in the can for storage.

## **6. Warranty**

- When worn under normal conditions and in compliance with the instructions for use, we guarantee a useful life of 2 years (legal warranty).
- In the event of a manufacturing or material defect, ABC will replace the nipples within two (2) years from the date of purchase. Please contact your specialist medical dealer for this.
- Please note that the warranty does not include the adhesive strength of the adhesive surface.
- The useful life of the adhesive surface, i.e. its adhesive power, varies and depends on the care of the product and, when worn on the skin, on the condition of the skin and the personal hygiene of the wearer.

## **7. Incident Reporting**

- Only serious incidents relating to the product, which lead to a serious deterioration of the state of health or death, must be reported to the manufacturer or competent authority of the EU Member State. Serious incidents are described in Article 2(65) of Regulation (EU) 2017/745 (MDR).

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ABC Breast Care GmbH