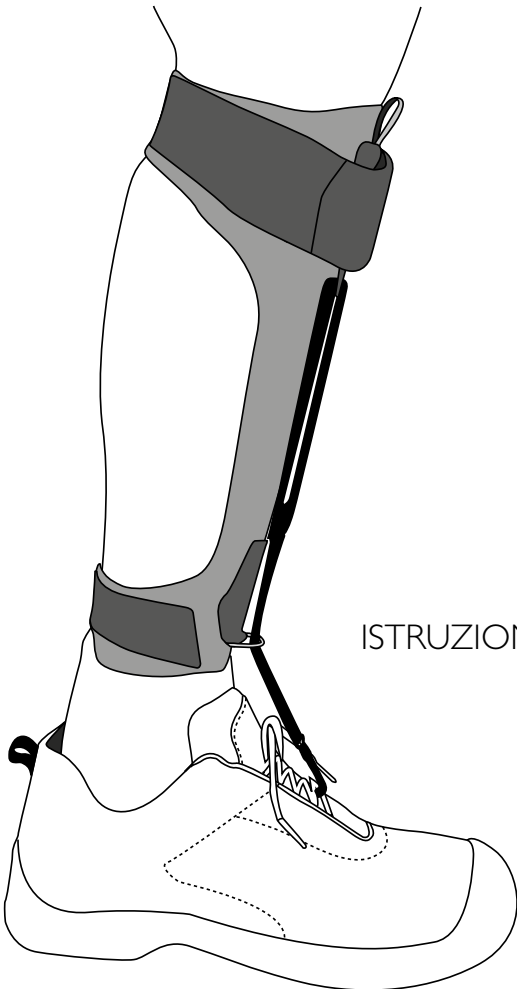
allard^{INT}

NAVIGAIT™

INSTRUKTION

INSTRUKSJON

INSTRUCTIONS FOR USE

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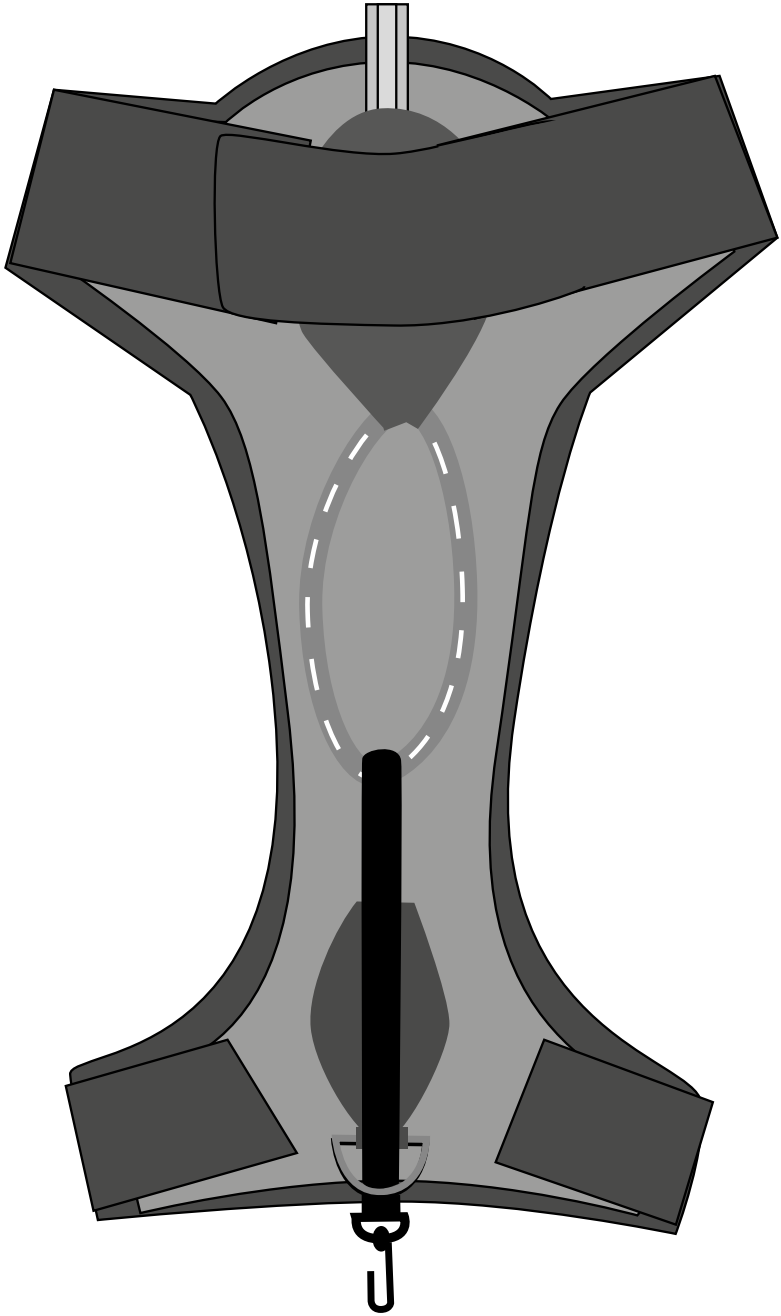
www.allardint.com

NAVIGAIT™

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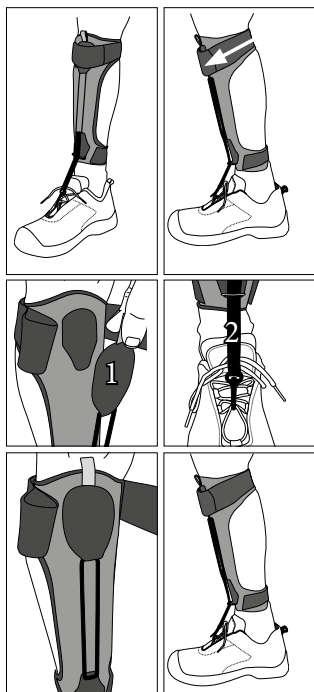
NAVIGAIT

Carefully read this instruction sheet before use!

Intended use and indications

Navigait is intended to support a foot when the ability to actively dorsiflex is reduced due to conditions such as Stroke, Multiple sclerosis, Post polio syndrome, Spinal Cord Injury.

Contraindications

Navigait should not be used by patients who present with spasticity or severe edema.

Adjustments

Adjustments to this product may only be done by professional healthcare providers. When needed, the plastic frame may be trimmed to size.

Warning

Do not pull the straps too tight, as this can cause poor circulation and/or skin irritation. Remove the brace immediately if you notice any changes in skin color or if you notice any tingling sensations, aching or chafing feeling. Check skin on a daily basis.

Material Specifications

Frame: Polyethylene

Padding: Outside material 100% polyester.

Core material polyurethane.

Elastic Coil: Rubber, contains Latex.

This product is not flame retardant.

Washing Instructions

Padding & Straps:

Remove the padding and straps from the frame.

Close all Velcro into the material prior to laundering.

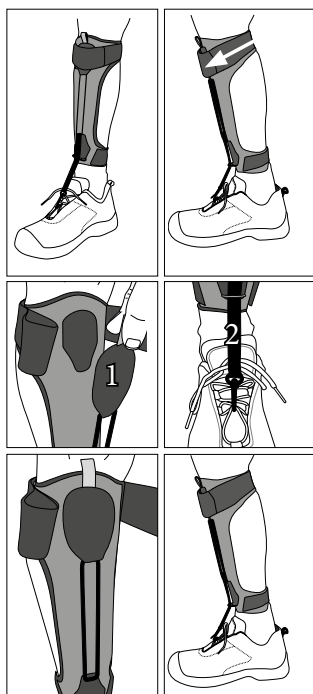
Machine washable in 40°C (104°F) gentle cycle. Use a mild detergent. Dry flat. Do not tumble dry or dry clean.

Plastic Frame:

The plastic frame can be rinsed in lukewarm water and cleaned with a mild soap. (Remove the elastic strap before cleaning). All parts must be dry before use!

Who to contact

Please contact your healthcare professional immediately in case of any discomfort, skin discoloration or irritation, or any signs of circulation problems.

Instructions for use

1. Check that the support is complete and without damage.
2. Position the brace so that the upper strap sits on the head of the calf muscle. Apply the strap at an angle from back to front as shown in the illustration. Leave open in front.
The lower edge of the brace should be seated comfortably above the ankle bones.
If needed the brace can be trimmed slightly in the upper and lower sections with scissors.
3. Adjust the tension (how much lifting affect you need) by either (1) moving the top velcro part, or (2) adjusting how far down the hook on the tension strap is located on the shoe lacing.
4. Secure the strap over the pad around the leg.

Care of your NavigAIT

When the brace is not being used, roll the plastic frame longitudinally. Wrap the straps around the top and the bottom to secure. This procedure will keep the plastic frame pre-shaped.

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
Item No.	Description	Size	Length Plastic section
28410 0010	NAVIGAIT™	X-Small	23 cm
28410 0011	NAVIGAIT™	Small/Medium	26,5 cm
28410 0013	NAVIGAIT™	Large/X-Large	30 cm
28412 0010	Extra coil w/hook	X-Small	
28412 0011	Extra coil w/hook	Small/Medium	
28412 0013	Extra coil w/hook	Large/X-Large	
28413 0010	Extra strong coil w/hook	X-Small	
28413 0011	Extra strong coil w/hook	Small/Medium	
28413 0013	Extra strong coil w/hook	Large/X-Large	
28415 0011	4-Foot Sock	Small	21 - 23 cm
28415 0012	4-Foot Sock	Medium	23 - 26 cm
28415 0013	4-Foot Sock	Large	26 - 29 cm

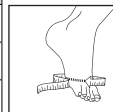NAVIGAIT™ 4-FOOT

 Camp Scandinavia AB, Karbingatan 38
 SE-254 67 Helsingborg, Sweden
 Phone +46 42 25 27 01

Camp Scandinavia, AUG, 2018©

allard | ^N | T

www.allardint.com

Tel +46 42 25 27 00
Fax +46 42 25 27 25

ALLARD INTERNATIONAL
 Camp Scandinavia AB
 Karbingatan 38
 SE-254 67 Helsingborg SWEDEN
 info@allardint.com