Instrucciones de adaptación · Anleitung · Tilpasning Instruktions · Οδηγίες εφαρμογής · Fitting instructions · Sovitusohjeet · Instruktions · Istruzioni per l'adattamento · Montage-instrukties · Tilpassingsinstruksjon · Instrukcja obsługi · Instrucciones de adaptación · Instruční manual · Uputstvo za instalaciju · Instrucciones de adaptación · Instruční manual · Uputstvo za instalaciju

Alza para Walker AW
Häckle ti walker AW
Trottervio Walker AW
Achilles wedges for walkers AW
Kantakilla Walkerin AW
Rialzo per Walker AW
Achilles wígegen voor Walkers AW
Häckle ti walker AW
Podpiórka do buta marszowego AW
Cunha para Bota Walker AW
Support de calcul pentru orteze Walker AW
Akilleskil för Walkers AW

PRIM

ES

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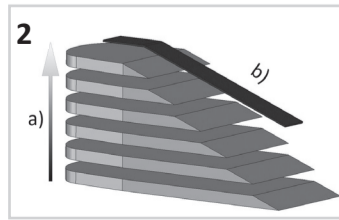
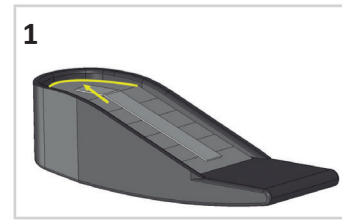
ENG

INDICATIONS

Postoperative Achilles tendon repair.

PRECAUTIONS

- Read these instructions carefully before using the orthosis. If you have any questions, consult your doctor or the shop where you purchased it.
- Only doctors are qualified to prescribe the orthosis and determine the duration of treatment.
- The fitting of the product should be carried out by your orthopaedic technician.
- For maximum effectiveness, you should respect the daily duration of use and follow the instructions of your doctor or orthopaedic technician.
- To ensure the full therapeutic effect, you must use the correct size.
- This orthosis must not be used for any other purpose than that prescribed. Do not let children play with this device.
- You must inform your doctor if you suffer from any skin conditions. Avoid using topical creams during treatment.
- In pressure bearing areas, the skin must not be damaged or hypersensitive. Do not wear the orthosis while having a bath, shower, etc.
- Regularly check that the wedges are properly adhered together and in good condition (no grease, cuts, tears, etc.).

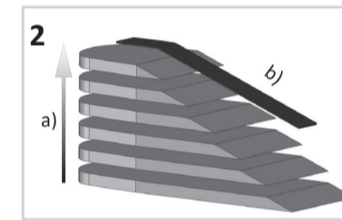
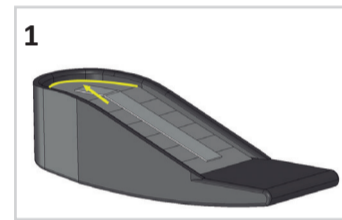
FITTING INSTRUCTIONS

- The wedges should be placed into the base of the walker orthosis. Make sure that the rounded part fits snugly against the back of the walker, as indicated by the arrow in the picture. The product features a Velcro strap on the top to which you can fasten the textile boot of the walker. Make sure that your foot is well positioned and secure before walking.
- To adjust the height, as per the instructions of your doctor:
 - Always remove layers from the bottom up. Each layer of the wedge represents an angle of approximately 4°, up to a total of approximately 20°, which is the full height of the product.
 - After removing the required number of layers, trim the excess Velcro.

MAINTENANCE OF THE ORTHOSIS - WASHING INSTRUCTIONS

Clean with a damp cloth. Do not use bleach. Do not iron. Do not dry clean. Do not tumble dry. It is advisable to keep the Velcro free from fibres, fluff, etc., to maintain its adhesive properties.

NOTE: Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State where the user and/or patient is located.

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