
OG - Oedema Gloves

EN	COMPRESSION GLOVE
DE	KOMPRESSIONSHANDSCHUH
FR	GANT DE COMPRESSION
ES	GUANTE DE COMPRESIÓN/PRESIÓN
IT	GUANTO DI COMPRESIONE
NL	COMPRESSIEHANDSCHOEN
FI	KOMPRESJONSTERAPI, HANSKE
SE	KOMPRESJONSHANDSKE
NO	KOMPRESJONSTERAPI, HANSKE
PL	REKAWICZKA UCISKOWA
GR	ΓΑΝΤΙ ΣΥΜΠΙΕΣΗΣ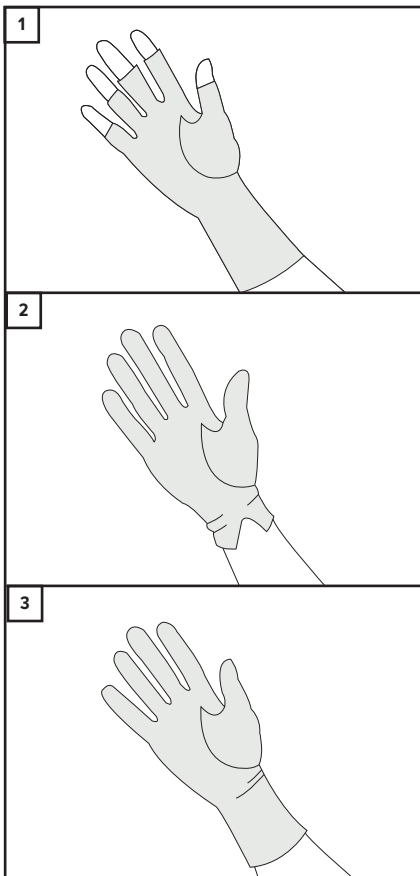

ENGLISH

Product Description

Therapeutic compression gloves. Available in open finger or full finger options.

Usage Information (See diagram)

Pull glove onto hand ensuring the chosen size is comfortable.

Indications for Use

- Provides gentle, even compression to help ease pain and prevent oedema.
- Reduces pain associated with arthritis or post hand injury pain and swelling.

Contraindications

- Do not use over open wounds.
- Any known allergy and/or hypersensitivity to any material listed in product composition.
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy.

Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

Washing and Care Instructions

1. Hand wash brace in cool water using mild detergent.
2. Rinse and dry flat.
3. Do not spin or tumble dry.
4. Do not use fabric conditioner at any time.
5. Check for any degradation that may affect use.

Product Family Composition

Nylon, Spandex.

Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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