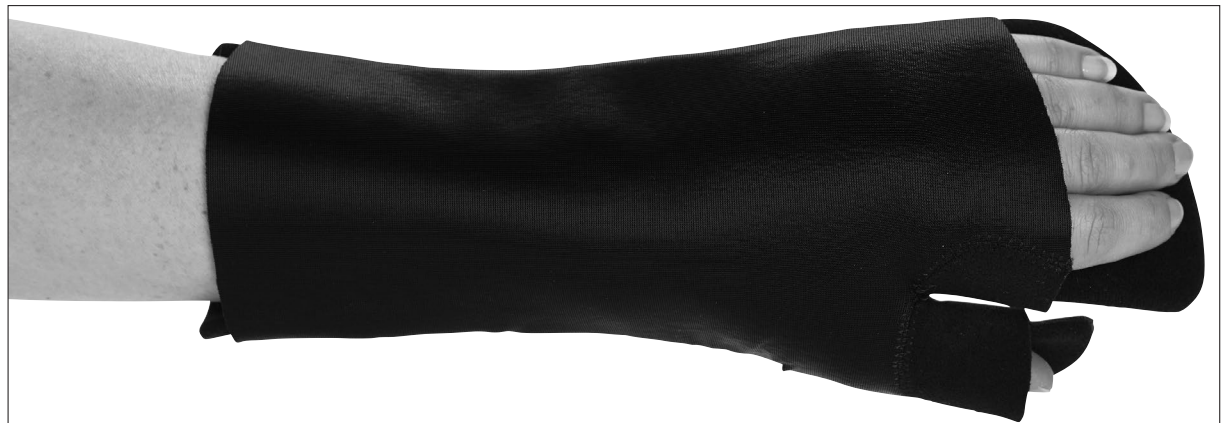
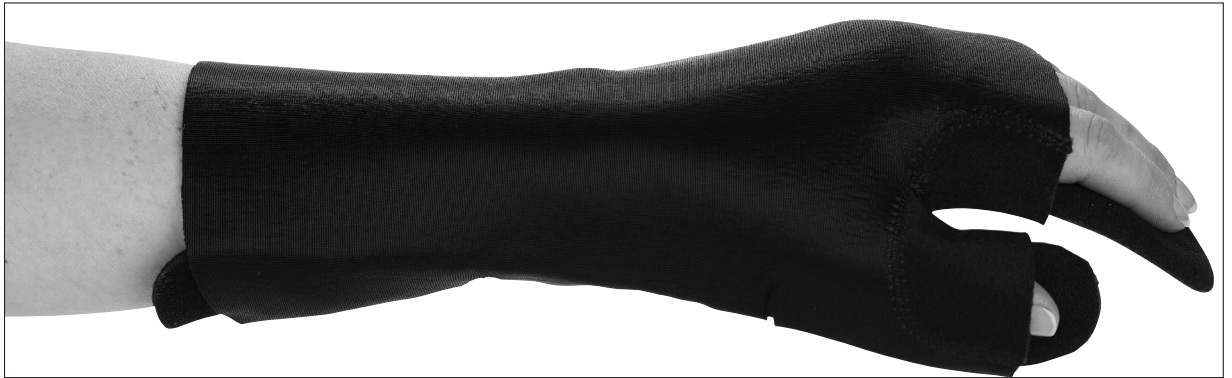
**28710**  
**RESTING SPLINT**

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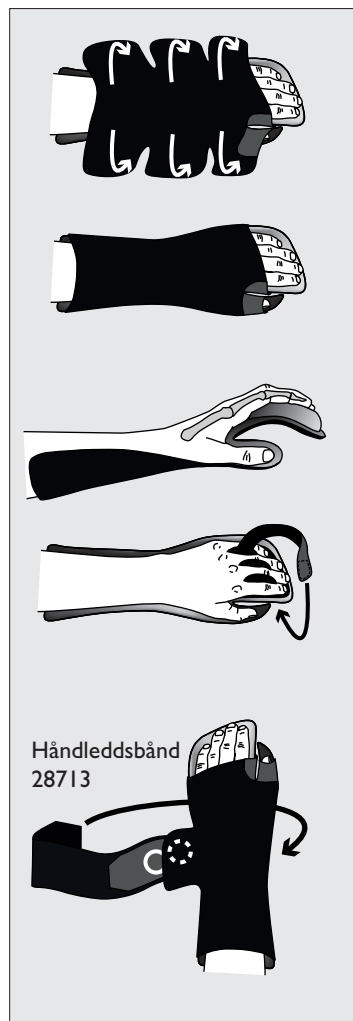
Håndleddsband 28713

## Intended Use

The product is intended for hands that need the support of an orthosis for rest or for stretching to maintain or increase the mobility of the hand. Good biomechanical positioning of the joints that can stimulate relaxation of the spastic hand and/or reduce pain. It can also be used as protection for patients who can benefit from the wrist in a neutral position to avoid lengthening of weak or paralyzed muscles. By the well distributed pressure on volar- and dorsal side of the hand and arm it can prevent or eliminate swelling. The product can be reshaped to "Position of Safe Immobilization" to be used after trauma and surgery. Single Patient-multiple use.

## Indications

Such as stroke, cerebral palsy, rheumatoid arthritis, plexus- and muscular dystrophy patients and radial nerve palsy.

## Contraindications

No known.

## Limitations

This product should only be altered by a medical professional person. The prescriber will determine the duration and conditions for use.

## Warning

Do not pull the straps too tight as this can cause poor circulation and/or skin irritation. Check the skin on a daily basis and if you see or feel any irritations or other problems, take it off immediately. If you have marks or chafing that do not reduce within a short time (30 minutes) contact your clinician immediately.

## 28710 S.O.T. RESTING SPLINT

Carefully read this instruction sheet before use!

## How to use the support

Check that you have the correct size. Your full hand and fingers should fit inside the orthosis. If the product have accessories mounted such as the finger divider, wedge or additional wrist strap, check that these are correctly positioned on the product. Avoid using oil-based lotions. If the patient sweats a lot, protect the orthosis by applying a stockinette or similar. Follow the fitters instructions for use.

## Material specifications

This product is made from Polyester, Nylon, Polyurethane and Spandex. Core of aluminum and polyethylene. It does not contain Latex. This product is not flame retardant.

## Wash instruction

Close all Velcro prior to laundering. Use a washing bag. Machine washable in 40°C (104°F) gentle cycle. Dry flat. Do not tumble dry or dry clean.

## Disposal

Disposal in accordance with local legislation.

## Information

Any serious incident that may occur in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.

## Who to contact

Contact your fitter or store where purchased if you have questions regarding the use of the product.

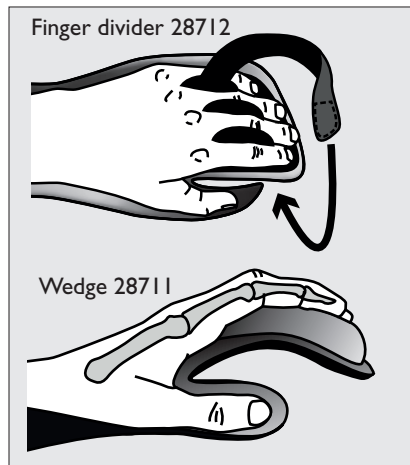
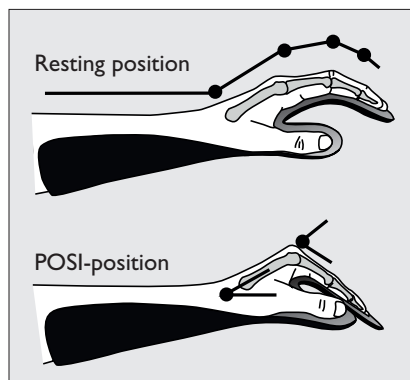
## Instructions for use

Check that the product is complete without damage!

### To the fitter:

To remould the orthosis in to POSI-position (Position of Safe Immobilization), the distal section of the splint must be adjusted. The area at the MCP-joints must be bent into between 60 - 80° of flexion and the area for the IP- and DIP-joints should be straighten to full extension.

1. The fitter should make sure that the orthosis have the correct shape and joint angles for the intended function. The orthosis have a core of aluminium that gently can be adjusted to the desired angle and shape for the wrist, thumb and fingers.

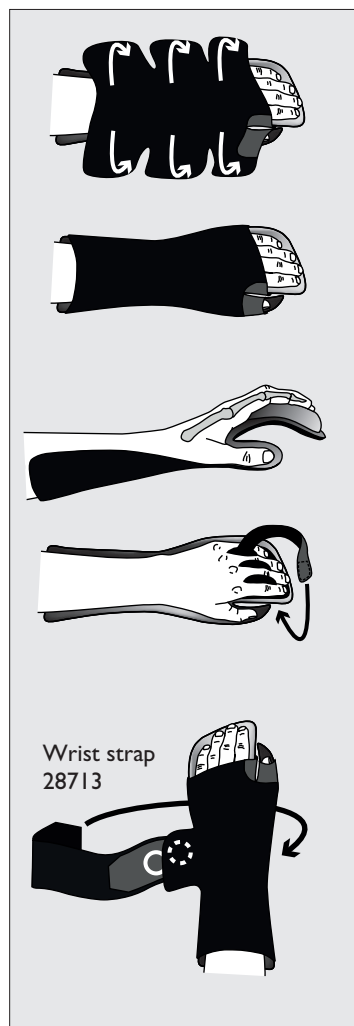
2. During fitting only open straps marked with white arrows in the picture to the right.

3. Position the hand into the orthosis. Make sure that the thumb is in a correct position. Start by applying the strap over the wrist. Then adjust the strap in sequences so that it gives a comfortable feel and does not give any proximal pressure.

An extra strap (28713) around the wrist can be added to give a better fixation over the wrist if needed due to increased spasticity.

If the orthosis is equipped with the finger divider (28712) or the wedge (28711), make sure that these are correctly positioned on the orthosis.

4. Make sure that the orthosis is comfortable and does not create any chafing or is too tight.

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
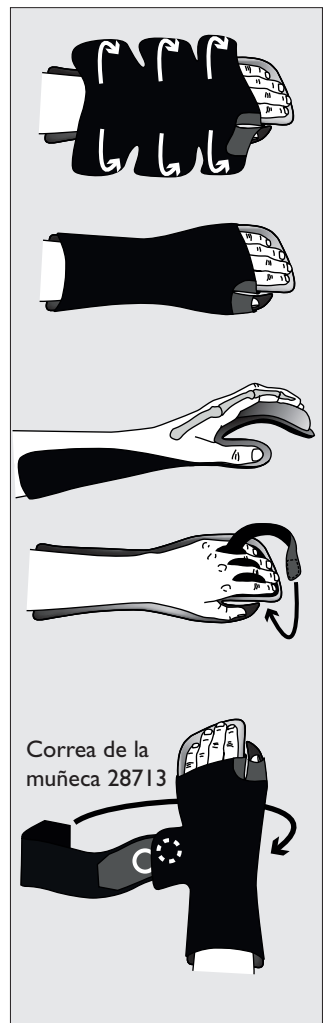
# 28710 RESTING SPLINT

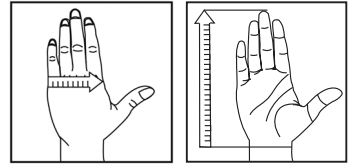Item No. Left	Item No. Right	Model	Size	MCP Width**	Length of hand ** (from wrist to end of fingers)
28710 1008	28710 2008	Resting Splint*	XXX-Small	5,5 cm	≤12 cm
28710 1009	28710 2009	Resting Splint*	XX-Small	≤ 6,5 cm	≤ 14 cm
28710 1010	28710 2010	Resting Splint*	X-Small	≤7,5 cm	≤ 16 cm
28710 1011	28710 2011	Resting Splint*	Small	≤7,5 cm	≤ 18,5 cm
28710 1012	28710 2012	Resting Splint*	Medium	≤ 8,5 cm	≤ 20 cm
28710 1013	28710 2013	Resting Splint*	Large	≤ 9 cm	≤ 21 cm
28712 0003		Finger divider	Pediatric		
28712 0005		Finger divider	Medium		
28712 0007		Finger divider	Large		
28713 0009		Wrist strap	XXXS-XS		
28713 0012		Wrist strap	Small-Large		
28714 1008	28714 2008	Additional covering strap	XXX-Small		
28714 1009	28714 2009	Additional covering strap	XX-Small		
28714 1010	28714 2010	Additional covering strap	X-Small		
28714 1011	28714 2011	Additional covering strap	Small		
28714 1012	28714 2012	Additional covering strap	Medium		
28714 1013	28714 2013	Additional covering strap	Large		
				Width	Height
28711 0009		MCP Wedge	XX-Small	65 mm	10 mm
28711 0011		MCP Wedge	Small	75 mm	15 mm
28711 0013		MCP Wedge	Large	90 mm	20 mm

\* includes splint + one covering strap

\*\* The ratio between hand and finger length varies, therefore, the dimension table should only be seen as a guide to the right size and fit.

Item No.	Description	XXXS	XXS	XS	S	M	L
28710	Resting Splint*	X	X	X	X	X	X
28714	Additional Covering Strap	X	X	X	X	X	X
28712	Finger divider	Pediatric		Medium		Medium/Large	
28711	MCP Wedge	XXSmall		Small		Small/Large	
28713	Wrist strap	XXXS-XS			Small-Large		

This tabel only serves as a guide between the accessories and the main product. Always try the size on the acutal user to ensure that the correct size is used.

Camp Scandinavia AB, Karbingatan 38  
SE-254 67 Helsingborg, Sweden  
Phone +46 42 25 27 01

<b>SVERIGE</b>	<b>DENMARK</b>	<b>SUOMI</b>	<b>NORGE</b>	<b>UK/IRELAND</b>	<b>USA</b>	<b>INTERNATIONAL</b>
Tel 042-25 27 01	43 96 66 99	09 350 76 30	23 23 31 20	+353 42 932 8177	888-678 6548	+46 42 25 27 01
Fax 042-25 27 25	43 43 22 66	09 350 76 388	23 23 31 21	+353 42 932 8182	800-289 0809	+46 42 25 27 25
info@camp.se	info@camp.dk	info@camp.fi	info@camp.no	info@allarduk.co.uk	info@allardusa.com	info@allardint.com

Support For Better Life • [www.allardsupport.com](http://www.allardsupport.com)