allard

Support for better life!

User Instructions

Selection Rigid

Art. No. 35202, 35203

ENGLISH

Indications / Intended Use

Is intended for mild to moderate arthritis in the first CMC-joint and protects the CMC-joints and the first MCP-joint. Rheumatic arthritis, after trauma, in case of hyper mobile thumb and wrist, after arthroplasty surgery, and for repetitive strain injuries. Single Patient-multiple use.

Contraindications

No known

Adjustments

This product should only be altered by a medical professional person.

Warnings

Make sure that the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

Material Specifications

This orthosis is made from polyester, nylon and elastan. This product is not flame retardant

35202, 35203

SELECTION THUMB ORTHOSIS RIGID

Carefully read this instruction sheet before use!

Washing Instruction

Close all Velcro prior to laundering. Machine wash in 40°C (104°F), gentle cycle. Use a washing bag. Dry flat. Do not tumble dry or dry clean.

Disposal

Disposal in accordance with local legislation.

Information


Any serious incident that may occur in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.

Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

Instructions for use

1. Check that the support is complete and without damage.
2. Attach the thumb strap and place the opening over the thumb with the orthosis towards the back of the hand.
3. Pull the strap around the wrist and secure.
4. Adjust the strap around the thumb so it follows the line indicated on the surface.
5. Adjust for comfort.

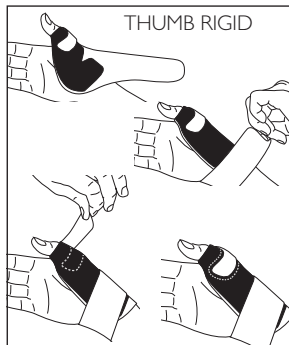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