ACTIVE X™

28055 Patella

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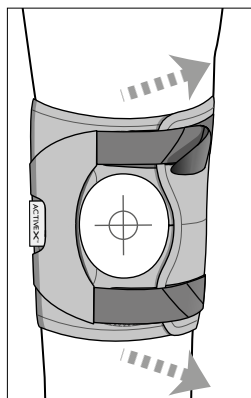
None known.

## Adjustments

This product should only be altered by a  
medical professional person.

## How to use the support

Check that you have the correct size. Some  
conditions can result in swelling that can mean  
variations in size. If this is the first time you are  
using a support it is important to start slowly.  
Use the support for a short period of time the  
first day, then increase the use gradually. The  
support should not be used overnight.

## 28055 ACTIVEX™ PATELLA

Carefully read these instructions before use!

## Warnings

Do not use directly on open wounds. Use only on dry  
skin. Do not use lotion on your skin. Make sure the  
straps are not too tight. If any sign of skin irritation  
occurs, immediately remove the support. Check your  
skin on a daily basis.

## Maerial Specifications

The main material in this product is made from 80%  
polyester/20% spandex. Other materials are 100% ny-  
lon and 100% silicon. Patella pad is plastic. This product  
does NOT contain Latex, and is not flame retardant.

## Wash Instruction

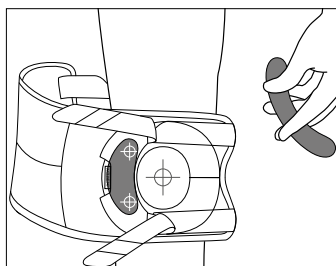
Close all Velcro prior to laundering. Machine wash in  
40°C (104°F). Use a washing bag. Dry flat. Do not  
tumble dry or dry clean. Remove the plastic insert  
prior to laundering.

## Who to Contact


If any problems occur, remove the support and contact  
your fitter or store where purchased.

## Instruktion

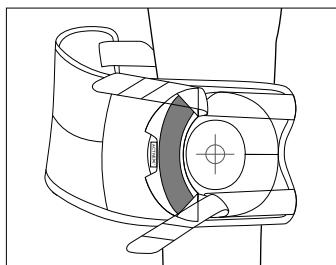
1. Check that the support is complete and  
without damage.


2. Open all Velcro straps. The support have  
a built-in pad and one removable extra plastic  
stabilizing support. The support can be used  
with or without the extra stabilizing support.

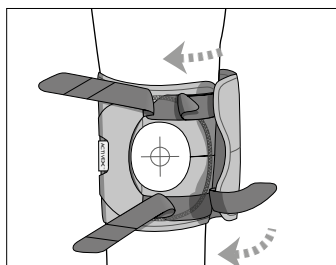3. With the knee slightly bent, position the opening  
over the knee cap (patella), with the built-in pad  
against the side of the knee cap. The pad should  
prevent the knee cap from moving into a wrong  
position.

4. With a light pressure over the pad - attach the  
two shorter straps.

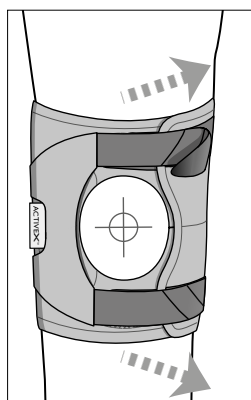5. Attach the other two straps to make a contra  
pressure to support the patella. The straps  
should not cross over the knee cap.

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Camp Scandinavia AB, Karbingatan 38 SE-254 67 Helsingborg, Sweden Phone +46 42 25 27 01