ACTIVE X™

28056 Sleeve

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ENGLISH

28056 ACTIVE X SLEEVE

Carefully read this instruction sheet before use!

Intended use

This product is intended to be used for light pain conditions in the knee.

Indications

Knee pain when loading, light ligament injuries, over use, light knee instability.

Contraindications

Severe knee injuries such as ACL/PCL ruptures, severe MCL/LCL injuries. Be observant when problem with swelling and/or poor blood circulation is present.

Adjustments

This product should only be altered by a medical professional person.

How to use the support

Check that you have the correct size. Some conditions can result in swelling that can mean variations in size. If this is the first time you are using a support it is important to start slowly. Use the support for a short period of time the first day, then increase the use gradually. The support should not be used overnight.

Warnings

Do not use directly on open wounds. Use only on dry skin. Do not use lotion on your skin. Make sure the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

Material Specifications

The main material is from polyester, spandex, nylon and 100% nylon/Latex free neoprene (SBR). Anti slide material from 100% silicone. This product does NOT contain Latex, and is not flame retardant.

Wash Instruction

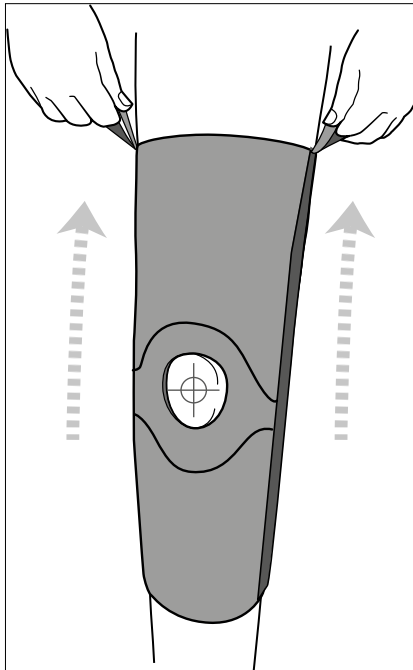
Close all Velcro prior to laundering. Machine wash in 40°C (104°F). Use a washing bag. Dry flat. Do not tumble dry or dry clean.

Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

Instructions for use

Check that the support is complete and without damage.

1. Pull the brace on to the leg.
2. Make sure that the brace is in the correct position with the kneecap centered in the opening.
3. Adjust and make sure the brace fits comfortably.

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Camp Scandinavia AB, June 2021 ©