**C**|S

**28679  
PATELLA  
CAMP**

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**Intended use/Indications**

This product is designed to achieve left or right lateral (or medial) stability of the patella. To be used for patella stabilization in conjunction with a rehab program for all non-surgical patella problems: Mis-alignments, chondromalacia patellae, dislocation of the patella or subluxation of the patella.

**Contraindications**

Excessive edema and circulation disorders.

**Adjustments**

This product should only be altered by a medical professional.

**Warnings**

Make sure the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

**Material Specifications**

Nylon, Polyurethan and Spandex. Steel stay and plastazote pad. This product is not flame retardant.

**Washing Instruction**

Attach all Velcro into material prior to laundering. Mashine wash warm. Hang dry. Do not tumble dry or dry clean.

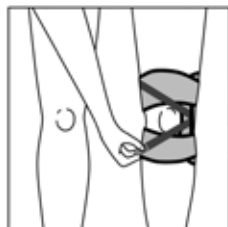
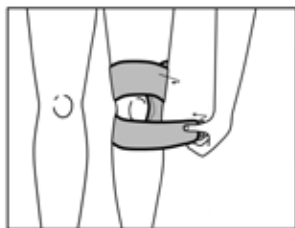
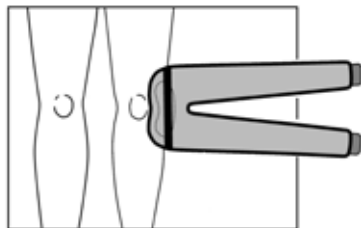
**Who to Contact**

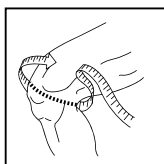
If any problems occur, remove the support and contact your fitter or store where purchased.

## Instruction

1. Check that the support is complete and without damage.
2. Apply from either lateral or medial side depending upon the direction of the instability.
3. Put the curved foam part snugly against the side of the patella to counter the direction of the instability.
4. Wrap the upper thigh strap firmly around the leg, above the knee.
5. Repeat the same for the lower strap.
6. Pull the small upper elastic strap over the top of patella and attach on the thigh strap.
7. Repeat for the lower elastic strap.
8. Make sure the straps provide the desired traction against the patella.

Storlek/Size	CM
Small/Medium	< 40 cm
Large/X-Large	> 40 cm

 Camp Scandinavia AB, Karbingatan 38  
SE-254 67 Helsingborg, Sweden  
Phone +46 42 25 27 01