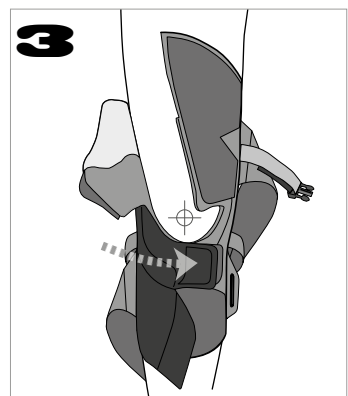
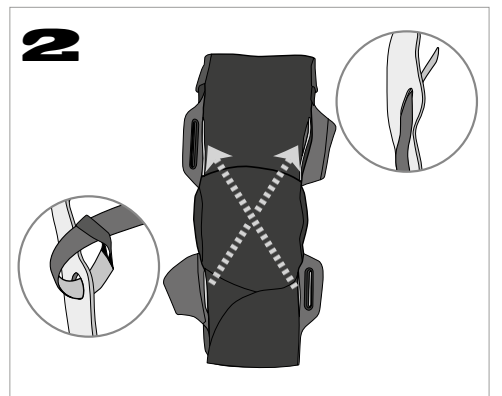
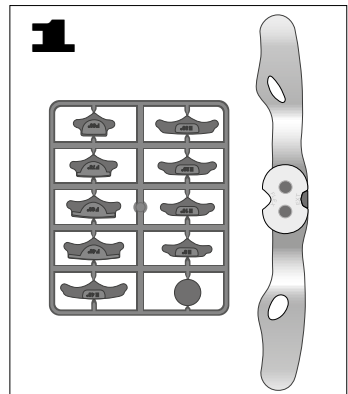
**35703**  
**Selection**  
**Optima WRAP**

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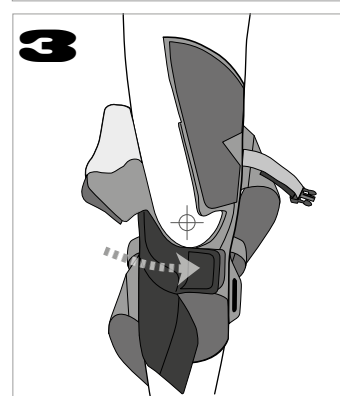
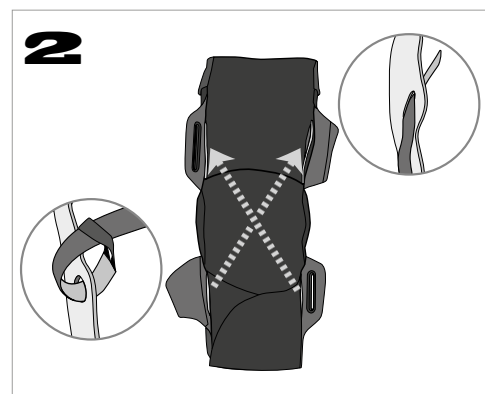
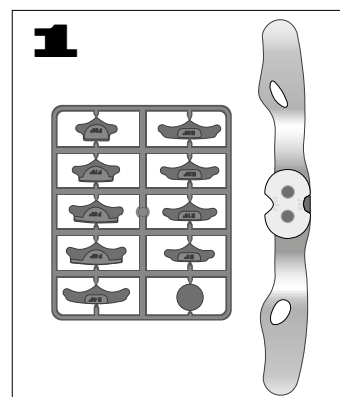
Carefully read this instruction sheet before use!

## Intended use

This support is intended for patients with various knee pain. It assists in reducing knee hyperextension, gives added M-L stability and can prevent tibia from moving forward when patients presents with weak ACL. The ROM-hinge can be utilized post surgery in stable knees. The support gives a light compression and keeps the knee warm. This support is only intended for one person use.

## Indications

Mild/moderate knee hyperextension, Weak ACL, Partial ACL rupture, Mild/moderate M-L Instability, Mild Osteoarthritis, Post surgery.

## Contraindications

Severe knee injuries such as ACL/PCL ruptures. Not to be used in contact sports or other high activities. Be careful: Severe swelling problems and blood circulation problems.

## Adjustments

This product should only be altered by a medical professional person, with experience from fitting knee braces.

## How to use the support

Check that you have the correct size. Some conditions can

result in swelling that can mean variations in size. If this is the first time you are using a support it is important to start slowly. Use the support for a short period of time the first day, then increase the use gradually. The support should not be used overnight.

## Warnings

Do not use directly on open wounds. Use only on dry skin. Do not use lotion on your skin. Make sure the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

## Material Specifications

This product is made from: Polyamid, polyurethane and spandex with Aluminum uprights. This product does not contain Latex, and is not flame retardant.

## Washing Instruction

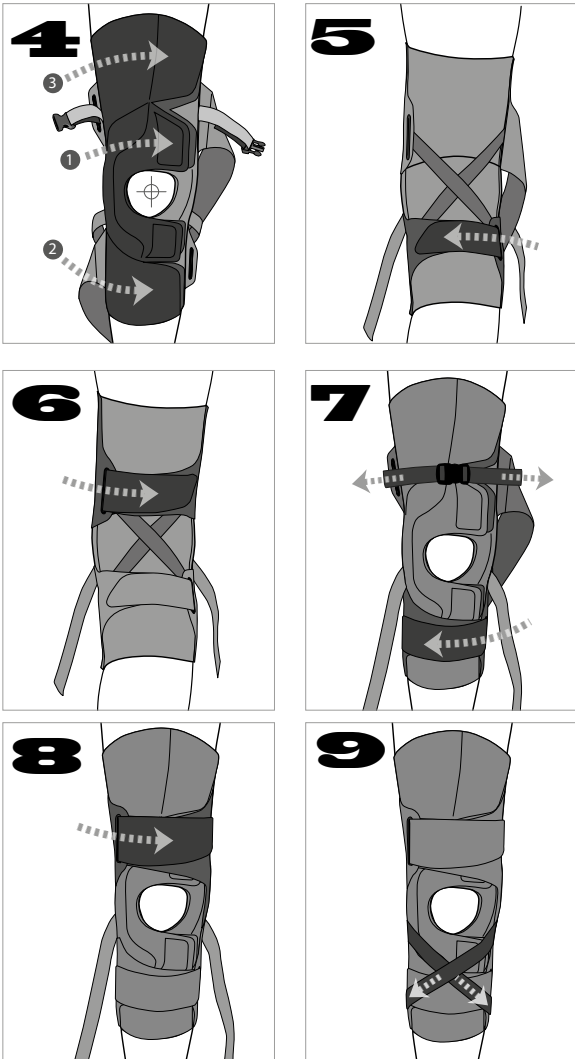
Close all Velcro prior to laundering. Machine wash in 40°C (104°F). Use a washing bag. Dry flat. Do not tumble dry or dry clean. Remove uprights prior to laundering.

## Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

## INSTRUCTIONS FOR USE

**Check that the support is complete and without damage.**

1. **This step should only be carried out by an authorized person.** Remove the joints from the brace. The brace is delivered with the joints set for 10° flexion. If different flexion/extension control is required, remove current flexion/extension stop and insert the desired one from those supplied with the brace, on both sides. If needed, shape the uprights to the contour of the leg. Avoid shaping close to the joint area. Re-insert the joints into the brace.
2. If the brace should be fitted with the hyperextension cross, this should now be mounted according to illustration no. 2. This is done with the uprights in the brace pockets.
3. Open all the Velcro closures. Position the brace onto the leg with the patella correctly positioned in the patella opening. Secure the strap under the knee.
4. Then close the remaining straps on the front as showed in picture 4.
5. Secure the straps on the back side of the leg (picture 5 and 6).
6. If the brace includes the hyperextension cross, adjust tension and snug this strap on the front of the thigh.
7. Then close the remaining straps.
8. Close the two straps under the patella. These two straps give a mild support to prevent tibia from sliding forward. If these straps are not needed - cut them off.
9. Adjust for comfort.

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Camp Scandinavia AB, Karbingatan 38