**35704  
Selection  
Patella**

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## Intended use

This product is intended for treatment of patella problems of various kinds. When positioning the patella pad on the side (medial/lateral) for treatment of patella luxation/subluxation. When positioning the patella pad from underneath for Patella tendon pain, Osgood Schlatter, Jumpers knee. The support gives a light compression and keeps the knee warm. This support is only intended for one person use.

## Indications

Patella luxation, Patella hypermobility, Patella tendon pain, Jumpers Knee, Osgood Schlatter, Patellofemoral Pain Syndrome.

## Contraindications

Non Known. Be careful: Severe swelling problems and blood circulation problems.

## Adjustments

This product should only be altered by a medical professional person.

## How to use the support

Check that you have the correct size. Some conditions

can result in swelling that can mean variations in size. If this is the first time you are using a support it is important to start slowly. Use the support for a short period of time the first day, then increase the use gradually. The support should not be used overnight.

## Warnings

Do not use directly on open wounds. Use only on dry skin. Do not use lotion on your skin. Make sure the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

## Material Specifications

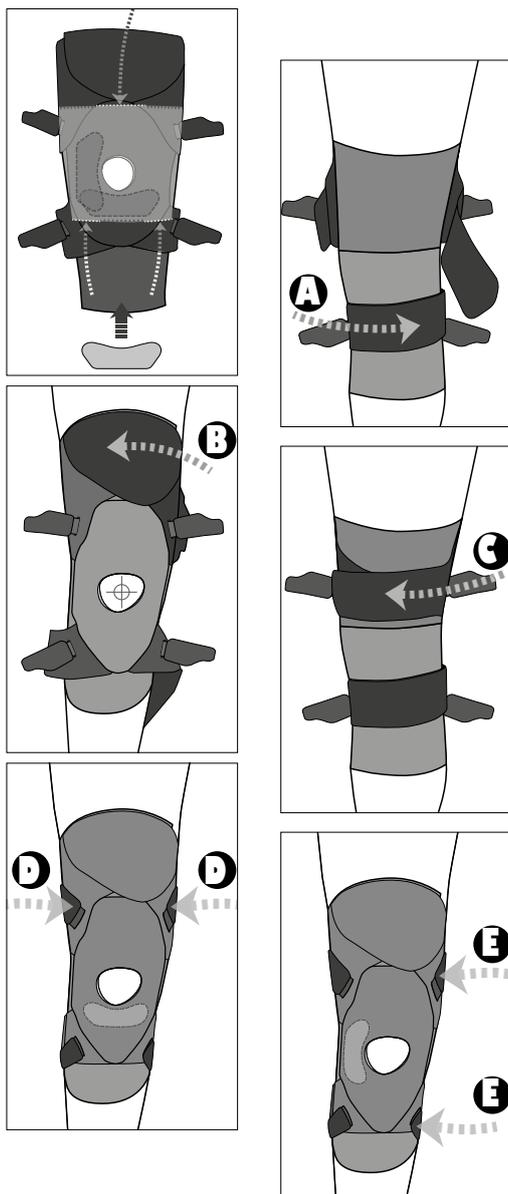
This product is made from: Polyamid, polyurethane and spandex with metal stays. This product does not contain Latex, and is not flame retardant.

## Washing Instruction

Close all Velcro prior to laundering. Machine wash in 40°C (104°F). Use a washing bag. Dry flat. Do not tumble dry or dry clean.

## Who to Contact

If any problems occur, remove the support and contact your fitter or store where purchased.

## INSTRUCTIONS FOR USE

1. Check that the support is complete and without damage.
2. Remove the Patella pad and insert in desired position (from the side or from under the patella). Loosen all the velcro straps.
3. Pull the sleeve up (use the two pulling loops placed on the inside of the upper edge) and make sure that the brace is properly positioned with the patella centered in the patella opening. Always make sure that the Patella pad is correctly positioned according to the instructions from the fitter.
4. Close the calf strap on the back side (A) and make sure the strap "sits" on top of the calf muscle to prevent from sliding down.
5. Close the front strap on the thigh (B). Then close the thigh strap on the back side of the leg (C).
6. Depending of the position of the Patella Pad - either pull the two tabs (D) to give support from under the patella, or the two tabs (E) to give support from the side of the patella.
7. Adjust for comfort.

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