**28050, 28051, 28052, 28053  
ACTIVE X SHORT/LONG**

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

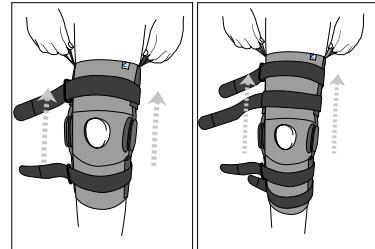
Wrap-around  
(28051 og 28053)

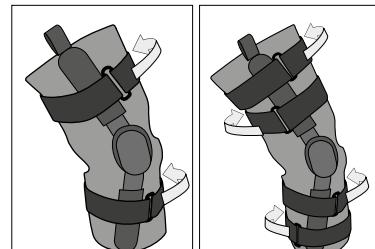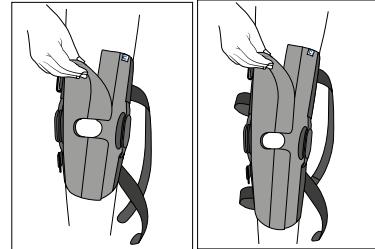**Intended use**

These products are intended for patients with various knee pain such as mild to moderate M-L instability. The brace gives a light degree of compression and keeps the knee warm. The brace is only intended for one persons use.

**Indications**

Mild to Moderate MCL/LCL Ligament injuries. Minor medial or lateral meniscus injuries. Mild osteoarthritis (the long models).

**Contraindications**

Total ligament ruptures (ML), ACL injuries, PCL injuries. Be careful: Severe swelling problems and blood circulation problems.

**Adjustments**

This product should only be altered by a medical professional person, with experience from fitting knee braces.

**How to use the support**

Check that you have the correct size. Some conditions can result in swelling that can mean variations in size. If this is the first time you are using a support it is important to start slowly. Use the support for a short period of time the first day, then increase the use gradually. The support should not be used overnight.

**Warnings**

Do not use directly on open wounds. Use only on dry skin. Do not use lotion on your skin. Make sure the straps are not too tight. If any sign of skin irritation occurs, immediately remove the support. Check your skin on a daily basis.

**Material Specifications**

The main material in this product is made from polyester, spandex and nylon. Antislide from silicone and uprights in aluminum. This product does NOT contain Latex, and is not flame retardant.

**Washing Instruction**

Close all Velcro prior to laundering. Machine wash in 40°C (104°F). Use a washing bag. Dry flat. Do not tumble dry or dry clean. Remove the aluminum uprights prior to laundering.

**Who to Contact**

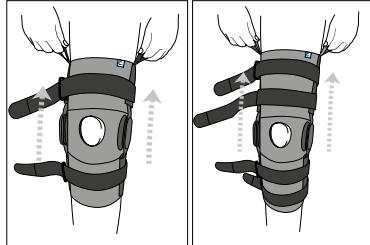
If any problems occur, remove the support and contact your fitter or store where purchased.



## INSTRUCTIONS FOR USE

**Check that the support is complete and without damage.**

Pull On  
(28050 och 28052)

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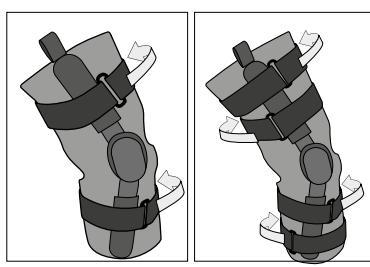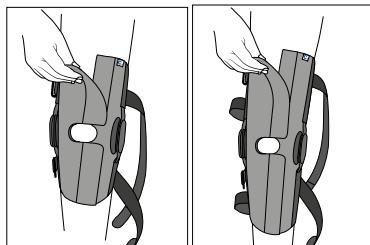1. **This step should only be carried out by an authorized person.** Remove the joints from the brace. The brace is delivered with the joints set for 10° flexion. If different flexion/extension control is required, remove current flexion/extension stop and insert the desired one from those supplied with the brace on both sides. If needed, shape the uprights to the contour of the leg. Avoid shaping close to the joint area. Re-insert the joints into the brace.

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Open all velcro straps and use the loops in the upper edge of the support to pull it on. Pull it over the knee and make sure that the knee cap is correctly positioned in the opening.

**3. Wrap-around version (28051 och 28053)**

Open all velcro straps. With the knee slightly bent, position the support behind the knee and close the two wide velcro sections on the front. Make sure that the knee cap is correctly positioned in the opening.

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