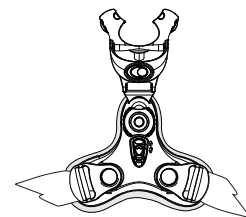
ASPEN VISTA® CTO UPGRADE KIT


CERVICAL THORACIC ORTHOSIS UPGRADE KIT
ZERVIKAL-THORAKALE ORTHESE ERWEITERUNGSATZ
KIT DE ACTUALIZACIÓN DE ÓRTESIS CERVICAL TORÁCICA
KIT DI AGGIORNAMENTO PER L'ORTESI TORACICA CERVICALE
KIT DE MISE À NIVEAU DES ORTHÈSES CERVICO-THORACIQUES
CERVICALE THORAKALE ORTHESE UPGRADE KIT
SADA PRO UPGRADE KRČNÍ HRUDNÍ ORTÉZY

OPGRADERINGSÆT TIL HALSHVIRVEL BRYSTKASSE ORTOSER
ΣΕΤ ΑΝΑΒΑΘΜΙΣΗΣ ΑΥΧΗΝΙΚΗΣ ΘΩΡΑΚΙΚΗΣ ΟΡΘΩΣΗΣ
CERVICALT OPGRADERINGSSET FOR BRYSTORTROSE
ORTEZA PIERSIOWO-SZYJNA ZESTAW ROZSZERZONY
KAULARANKA-RINTAKEHÄORTOOSI PÄIVITYSPAKKAUS
UPGRADERINGSATS FÖR CERVIKAL THORAKAL ORTOS
КОМПЛЕКТ ДЛЯ РАШШИРЕНИЯ ШЕЙНОГО КОРСЕТА
KIT DE ATUALIZAÇÃO DE ÓRTESE CERVICAL TORÁCICA

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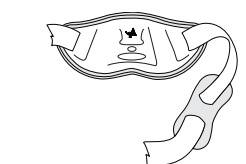MD MEDICAL DEVICE

SINGLE PATIENT - MULTIPLE USE

CAUTION

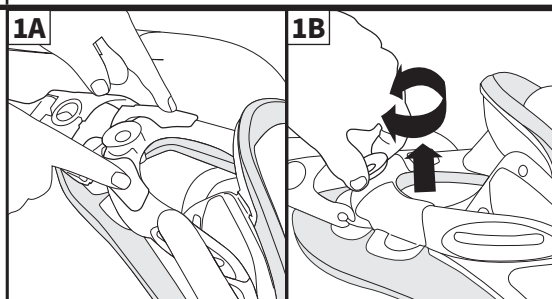
EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands


Pat. aspenmp.com/patents

Aspen Medical Products, LLC
6481 Oak Canyon, Irvine, CA 92618-5202 USA
aspenmp.com • +1-800-295-2776 • +1-949-681-0200


©2021 Aspen Medical Products, LLC. All Rights Reserved.
AC0129D revised 05/2021


ENGLISH
1. VISTA ADJUSTMENT
A. Using your index fingers, apply Vista CTO Anterior Panel to the Vista Collar front.
B. Pull and twist dial to adjust the height on the Vista Collar (see Vista Instructions at www.aspenmp.com).
2. VEST ADJUSTMENT
A. Push the button and pull to separate, disengaging the angle adjustment yoke.
B. Determine the desired custom angle.
C. Squeeze yoke back together to secure the custom angle.
D. Push down on the secondary lock below the primary yellow lock. Slide the primary yellow lock down to enable vertical height adjustment.
E. Turn the dial until the vest sits at the desired position. When fitting a supine patient, the vest should be positioned as low as possible.
F. Slide the yellow button up to engage the primary lock.

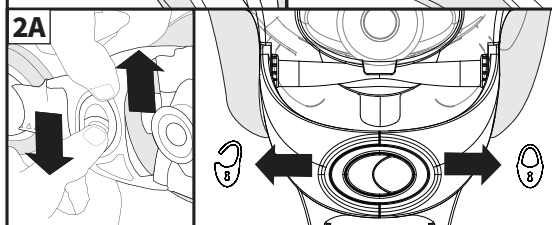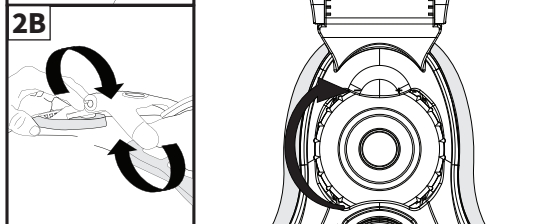
3. POSITION BACK PANEL
A. Slide back panel under lumbar spine.
B. Attach the back panel clips to the sides of the vest.
C. Pull the straps evenly until the back panel is tightened. Adjust the strap padding for comfort.


INDICATIONS
Decompressive procedures, Fracture management, Postsurgical stabilization, Whiplash, Sprains/strains, Cervicogenic headache, Cervical disc syndrome, Musculoskeletal pain, Radiculopathy.
CONTRAINDICATIONS
Hypersensitivity or allergies to any of the materials from which the brace is made. Swelling of the lymphatic tissue caused by circulatory disorders. Patients who are not allowed to wear a brace according to medical instructions.
PRECAUTIONS
READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations. If you suffer from a serious medical condition, we strongly suggest that you consult with a licensed health care professional before using this product. Proper fitting is required for this product to be effective. Under some circumstances, this product may be prescribed by a physician. Please see the limited warranty for further information.

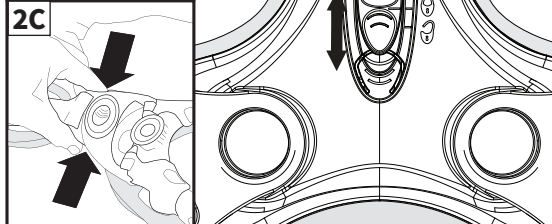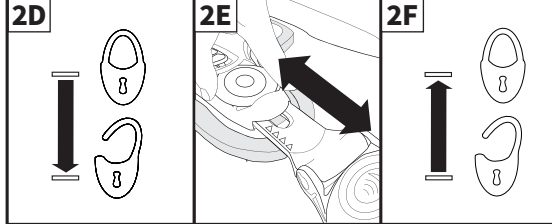
CARE INSTRUCTIONS
Hand wash only at 30° C. Wash with mild detergent; Air dry; Do not use bleach or other chemicals.
MATERIALS
ABS, Nylon, HDPE, Polycarbonate, Delrin, Polyurethane, Cotton, Aluminum.
NOT MADE WITH NATURAL RUBBER LATEX.

DISPOSAL
Product may be safely disposed in accordance with local laws.
REPORTING
Please report any serious incident involving the use of this device to both the Competent Authority in your state and to the manufacturer (listed on this document).
LIMITED WARRANTY
Aspen Medical Products, LLC, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products, LLC in the event of breach of warranty shall be to repair or replace the defective product or parts).

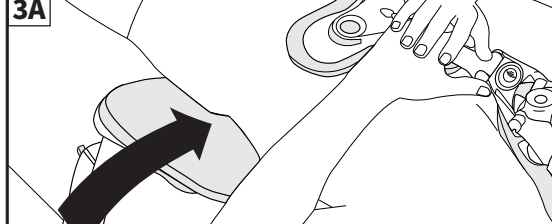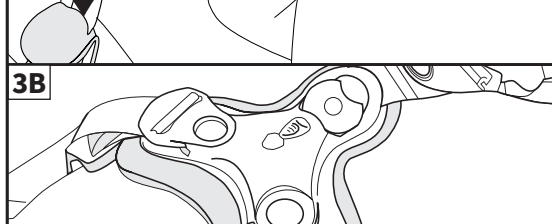Aspen Medical Products, LLC shall have no obligation under this limited warranty in the event:
(a) The product was not purchased from Aspen Medical Products, LLC or through its authorized channels of distribution;
(b) The product is altered;
(c) Any parts not supplied by Aspen Medical Products, LLC are inserted into the product; or
(d) The product is not used in accordance with the Aspen Medical Products, LLC Instructions for Use.
THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS, LLC SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT, SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS, LLC.
This warranty gives you specific legal rights and you may have other rights which vary from location to location.
A listing of authorized distributors in your area is available upon request.

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