CAMP[®]

BostonBrace

RIGIDA SPINALA ORTOSER

Boston Skolios Modul - originalet sedan 1972

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Brace Wear Control of Curve Progression in Adolescent Idiopathic Scoliosis

D.E.Katz, BS, CO, et al. The Journal of Bone and Joint Surgery, 2010;92:1343-52

The Boston brace is an effective means of controlling curve progression in patients with adolescent idiopathic scoliosis when worn for more than twelve hours per day.

Effects of Bracing in Adolescents with Idiopathic Scoliosis

S.L.Weinstein, MD, et al. The New England Journal of Medicine, Sep. 19/2013

In conclusion, bracing significantly decreased the progression of high-risk curves to the threshold for surgery in patients with adolescent idiopathic scoliosis. Longer hours of brace wear were associated with greater benefit.

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**Outcome of treatment for scoliosis.
A multi-disciplinary follow-up at least 20 years
after surgery or brace treatment in patients
with adolescent idiopathic scoliosis with
comparison to a matching control group**

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The study aims where to evaluate the outcome in terms of radiological appearance, degenerative changes, pulmonary function and different aspects of health related quality of life at least twenty years after completed treatment, with brace or surgery, in 283 consecutive patients.

Result; The majority of the patients were well-functioning physically, psychologically and socially even if some had developed an increase of the curve (4%) or admitted problems known to be associated with the disease.

**Effect of a Soft Boston Orthosis on Pulmonary
Mechanics in severe Cerebral Palsy**

M.T. Leopando, MD et al., Pediatric Pulmonology 28:53-58, 1999

Spinal braces such as the Soft Boston Orthosis (SBO) help stabilize scoliosis and improve sitting, positioning, and head control in individuals with cerebral palsy. However, their impact on pulmonary mechanics in this population has not been studied. We examined the effect of a Soft Boston Orthosis on the pulmonary mechanics and gas exchange in 12 children and young adults (5-23 years of age) with severe cerebral palsy.

"We conclude that the application of a Soft Boston Orthosis does not impact negatively on pulmonary mechanics and gas exchange in young people with severe cerebral palsy".

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**Soft Boston Orthosis in Management of
Neuromuscular Scoliosis:
A Preliminary Report**

M.Letts, MD., et al., Journal of Pediatrics 12:470-474, 1992

Collapsing neuromuscular spinal curvature is extremely difficult to manage with standard rigid orthoses owing to skin breakdown and patient intolerance. Experience with a new, more malleable type of spinal brace (the "soft Boston orthosis") has been very rewarding in 55 children with neuromuscular scoliosis with an average curvature of 42 degrees. Patient tolerance was improved in 38%, patient handling was facilitated in 68%, and postural positioning was enhanced in 79%. Although improvement in the scoliosis averaged only 15 degrees, the major advantage of this orthosis was a marked improvement in sitting stability in 80% of patients.

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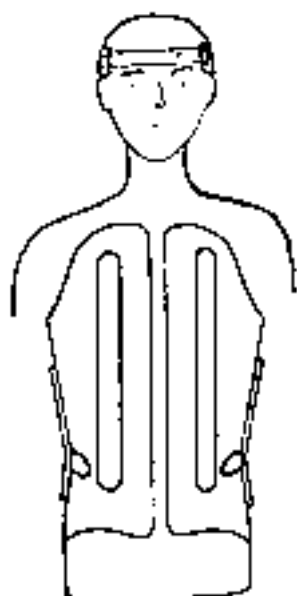
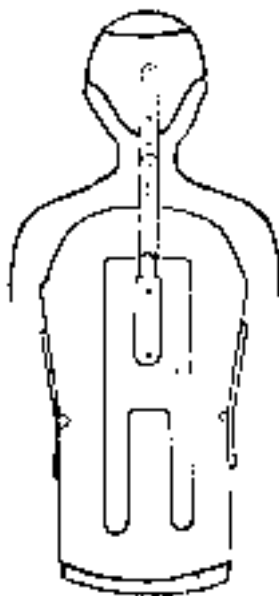
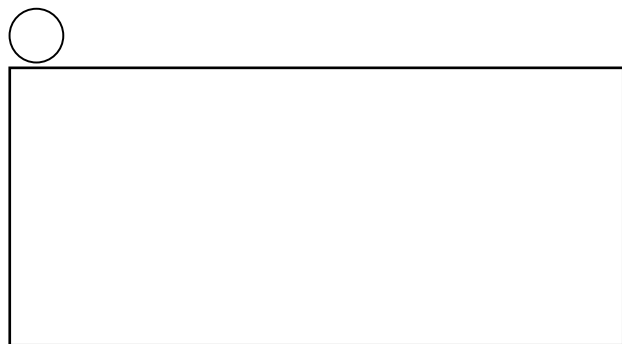
Order form Boston Braces Scolios/BOB/Flexaform/SoftBrace

Send the completed form to
info@camp.se

Customer: _____
 Prescriber/Orthotist: _____
 Patient ID: _____
 Age: _____ Height: _____ Gender: _____
 Diagnosis: _____

P.O.# _____
 Date: _____
 Delivery date: _____
 Ship To: _____

Has the patient used a Boston Brace before: Yes No
 Camps order no/your pat id/order no.: _____

Brace Type:

Scoliosis
 BOB (Lined) BOB (Unlined)
 Soft Body Jacket (removable stays)
 Soft Body Jacket (permanent stays)
Based on anatomical measurements
 Soft Body Jacket (permanent stays)
Based on individual measures, enclose Stay Chart
 Soft Body Jacket with Internal Frame
 Soft Body Jacket with External Frame
 Body Jacket (Lined) Body Jacket (Unlined)
 Boston LITE

Brace Design:

Degree of Lordosis: _____
 Brace to be made to Measurements Cast Scan**
 Opening: Anterior Posterior
 Bivalve Left Lateral
 Right Lateral
 Overlap: YES NO
 Finished to first fitting: YES NO
 ** a carving charge will be added to the order; see article number 29737

Material & Thickness (if non-standard required):

Colour: Soft Body Jacket and Boston LITE

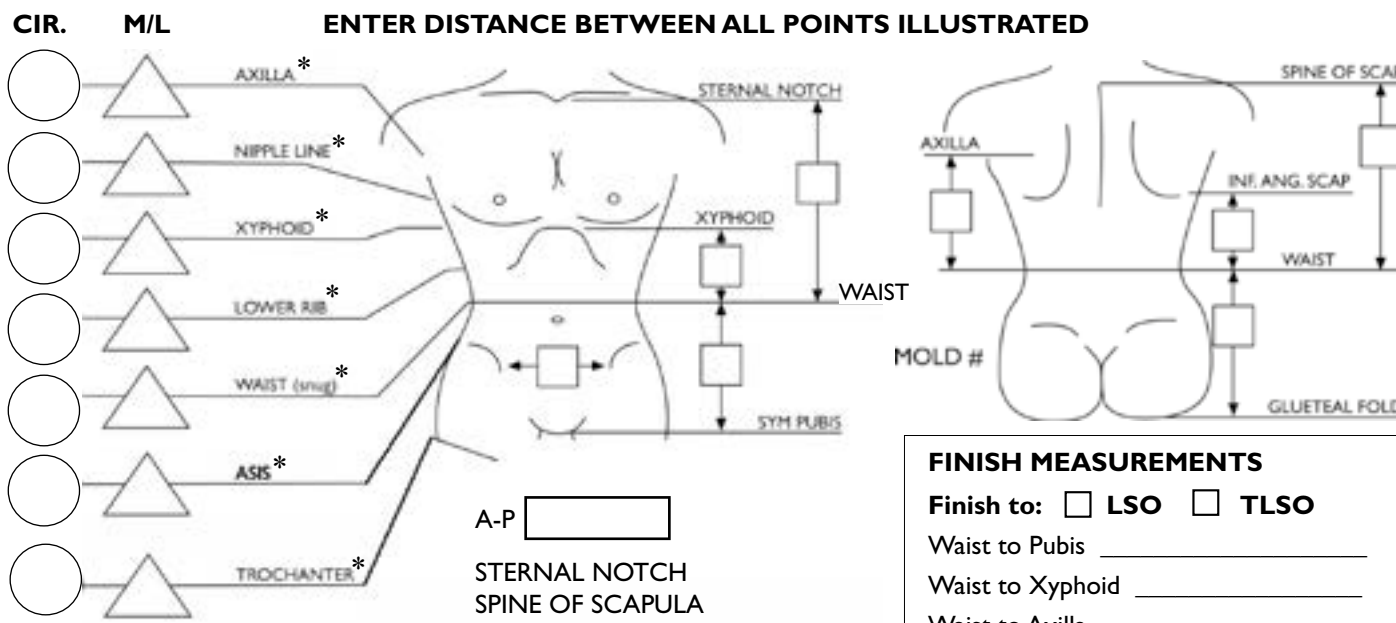
White Orange Blue
 Pink Purple

Other braces, including SBJ External frame:
 All over transfer pattern: _____
 Single placement transfer: _____

Should patient's belly size be taken into consideration when producing the brace?
 NO Yes (if yes, please enclose a photo where possible)

Are breasts built into the brace? Yes No
 Bra cup size _____ Height from waist to nipple line _____

Remarks _____

*Both Cir. & M/L are mandatory for Boston Scolios Scan-to-Fit

FINISH MEASUREMENTS

Finish to: LSO TLSO

Waist to Pubis _____
 Waist to Xyphoid _____
 Waist to Axilla _____
 Waist to Sternal Notch _____
 Waist to Inferior Angle _____
 Waist to Spine of Scapula _____
 Waist to Seat _____

Signed _____

Order form Boston Scoliosis Night Brace

P.O.# _____

Date: _____

Delivery date: _____

Ship To: _____

Customer: _____

Patient ID: _____ Age: _____ Height: _____ Sex: _____

Diagnosis: _____

Have patient used Boston Night Brace before: Yes No Date for last order/order no.: _____

Orthosis Design

Provide major curve: Left Right

Brace bending to: Left Right

Cobb Angles: _____ Lumbar _____ Thoracic _____

Brace Options: *Opening is always anterior*

Removable liner Permanent liner

Finished: Yes NO

Options: Transfer Pattern Number: _____

Curve Description

Thoracic curve: Left Right

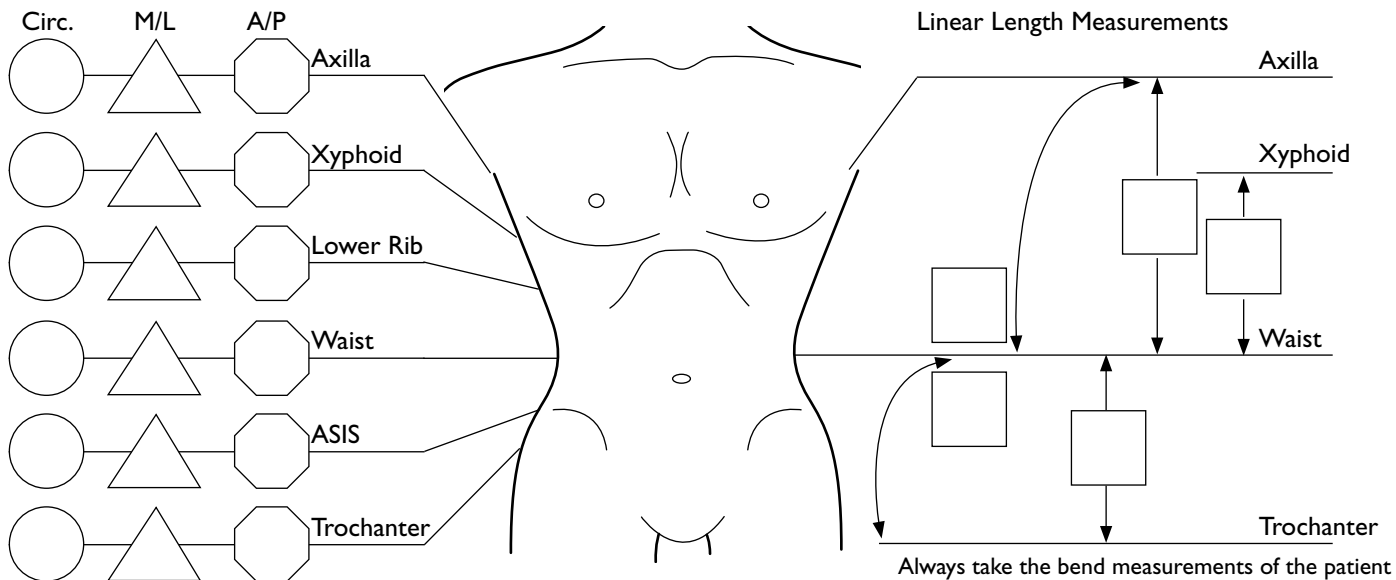
Lumbar curve: Left Right

Thoracic apex: Th- _____ Lumbar apex L- _____

Curve Type

Thoracic Lumbar

Thoracolumbar Double

Always take the bend measurements of the patient in the opposite direction of the primary curve.

NOTE: X-rays together with measurementform must be sent by e-mail.

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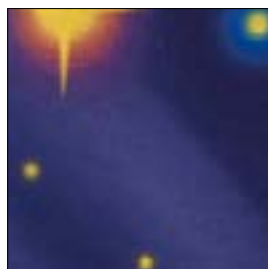
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
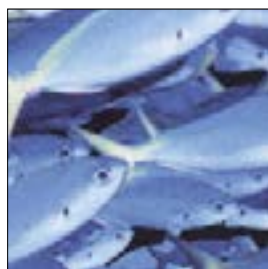
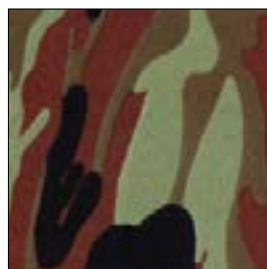
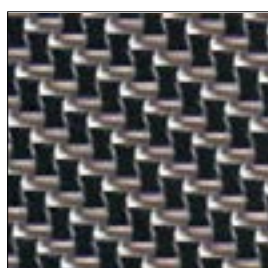
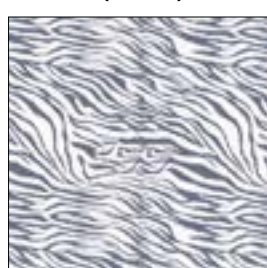
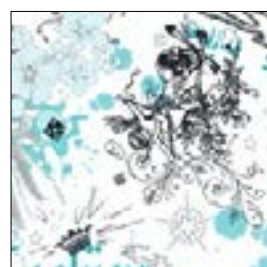
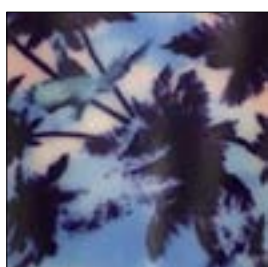
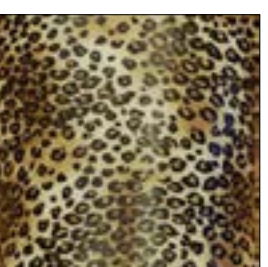
Boston Brace Transfer - All Over Pattern

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