C | **S**

23550 Regular Blue
23555 Regular Grey
23560 Regular Beige

NAVY X™

23550/23555/23560 NAVY-X

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23550/23555/23560 NAVY-X

Carefully read this instruction sheet before use!

ENGLISH

Indications / Intended Use

Navay-X is a lumbosacral back support to be used for light back pain and back weakness. It can also be used to support the spine during heavy lifting.

Contraindications

None found.

Adjustment

This product should only be altered by a medical professional person.

Warning

Do not pull the straps too tight, as this can cause poor circulation and/or skin irritation. If you feel any irritations or other problems, take it off immediately. Check skin on a daily basis.

Material specifications

78% Polyester, 22% Lycra, plastic stays. This product does not contain Latex. This product is not flame retardant.


Wash instruction

Close all Velcro prior to laundering. Machine washable in 40°C (104°F). Dry flat. Do not tumble dry or dry clean.


Who to contact

If any problems occur please contact your orthotist or fitter.


Instructions for use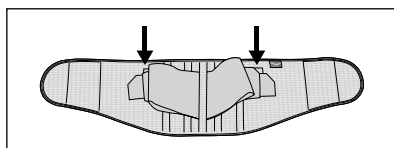


1. Start by placing the Velcro stabilizing straps on the side tabs.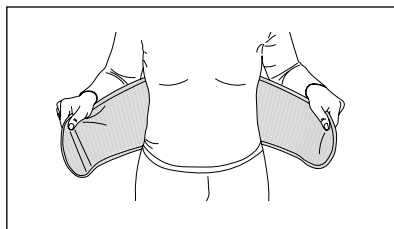


2. Put the brace on and close the front Velcro. Make sure it is properly positioned and centered in the lower back area.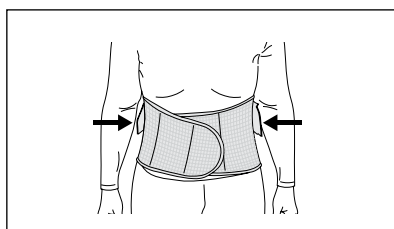

3. Adjust until it feels comfortable.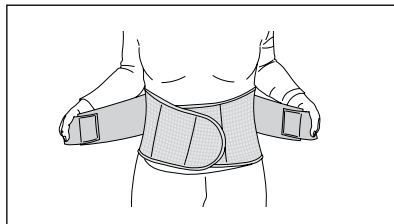

4. Take the extra stabilizing straps from the Velcro tabs and pull to front. Secure on the front Velcro.

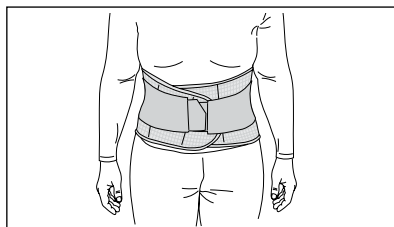5. To remove: First open the stabilizing straps and place them on the side tabs.

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