ASPEN VISTA® LOWER SPINE

- LUMBAR SACRAL ORTHOSIS
- LUMBAL-SAKRALE ORTHESE
- ÓRTESES LUMBOSACRA
- ORTESI SACRO-LOMBARE
- ORTHESE LOMBO-SACRÉE
- LUMBALE SACRALE ORTHESE
- BEDEBNÍ KRÍŽOVÁ ORTÉZA
- LUMBAL SAKRAL ORTOS
- ΟΣΦΥΪΚΗ ΟΡΘΩΣΗ ΟΣΤΕΟΥ
- LUMBAL KORSRYGGORTROE
- ORTEZA LEDŹWIOWO-KRZYŹOWA
- LUMBAL SAKRAL ORTOS
- LANNERANGAN SAKRAALINEN ORTOOSI
- LUMBAL SAKRAL ORTOS
- ОРТЕЗ ДЛЯ ПОЯСНИЧНО-КРЕСТЦОВОГО ОТДЕЛА ПОЗВОНОЧНИКА
- ÓRTESES SACRO LOMBAR
- 腰骶矫形器









ASPEN VISTA 627 LUMBAR

ASPEN VISTA 631 LSO LO PRO

ASPEN VISTA 637 LSO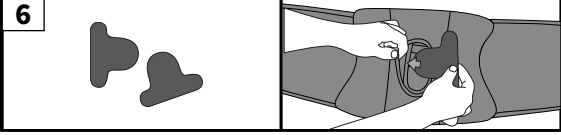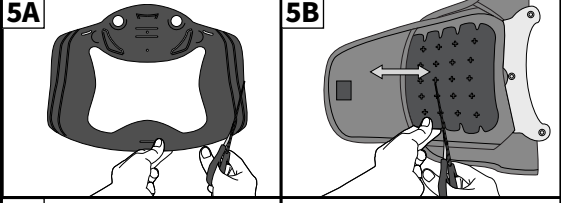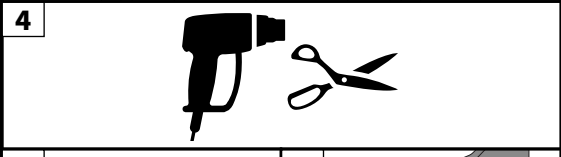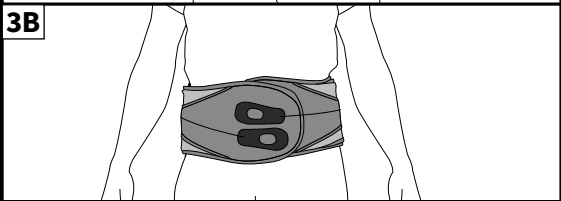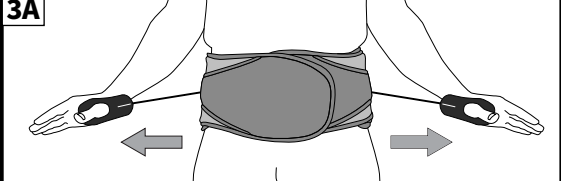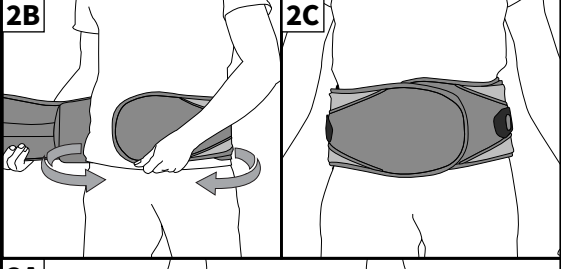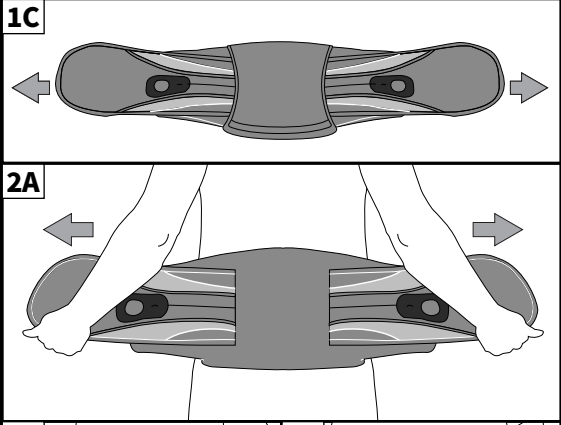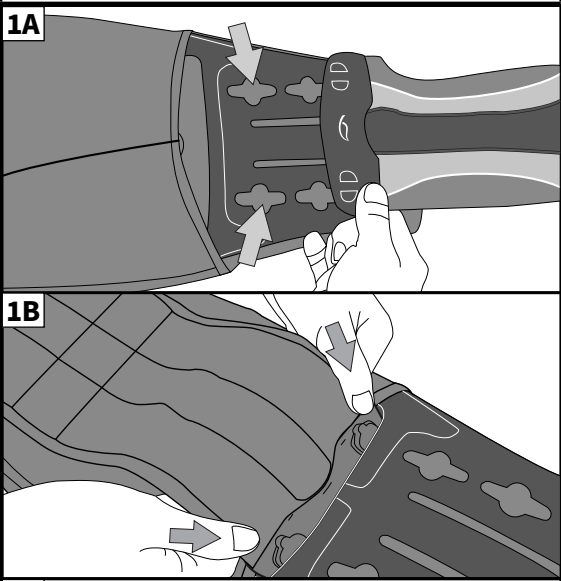

ENGLISH

1. BELT SIZING
A. Fit male rivets of belt through proper sizing adjustment holes in sizing mechanism.
B. For smaller sizes, slide side panel through sliding adjustment slot and then fit rivets through proper adjustment holes from the under side.
C. Once proper size is achieved, pull taught to lock rivets in place.

Size	Circumference
X-SMALL	26 – 31 in 66 – 78 cm
SMALL	30 – 37 in 76 – 94 cm
MEDIUM	36 – 43 in 91 – 109 cm
LARGE	42 – 49 in 107 – 124 cm
X-LARGE	48 – 55 in 122 – 140 cm
XX-LARGE	54 – 60 in 137 – 152 cm

2. PLACEMENT. Center back panel in the middle of the back as low as is comfortable.
B. Wrap left side of belt tightly to the front of the body.
C. Repeat with right side, and while inhaling, press the hook loop material to achieve a snug fit.
NOTE: Be sure top and bottom of brace is snug. If needed, angle ends to create a snug fit.

3. TIGHTENING
A. Pull strongly away from body. Tighten more than you need and then loosen to achieve a proper fit. Pull left tab to tighten lower, pull right to tighten upper as needed.
B. Mate the pull tabs to loop material to maintain compression.

4. MODIFICATION OPTIONS
This product may be substantially modified by an individual with expertise. For more information, please visit www.aspenmp.com/education/product-modifications.

5. MODIFY PANELS
A. When necessary, reduce overall width of posterior panel by removing from sleeve and utilize trim lines to cut to appropriate size.
B. Lateral panel placement can be adjusted in the sleeve or trimmed to ensure panels sit at midline on the patient.

6. LORDO PADS
Remove Lordo Pads, if needed, for personal comfort.
INTENDED USE
To provide motion restriction of the lumbar-sacral spine.

INDICATIONS
Postoperative support, Deconditioned trunk musculature, Post laminectomy syndrome, Surgical Decompression procedures, Non-surgical Decompression procedures, Degenerative disc disease, Bulging or herniated disc, Non-specific low back pain, Chronic low back pain, Fracture management, Spondylolysis, Spondylolisthesis, Spinal stenosis, Ligament sprains/ Muscle strains, Spondylolysis (Osteoarthritis/Degenerative Joint Disease), Facet syndrome, SI dysfunction, Sacroiliitis, Radiculopathy, Injections, Radiofrequency Ablation, Lumbar Multifidus Muscle Atrophy.

CONTRAINDICATIONS
Hypersensitivity or allergies to any of the materials from which the brace is made. Swelling of the lymphatic tissue caused by circulatory disorders. Patients who are not allowed to wear a brace according to medical instructions.

PRECAUTIONS
READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations. If you suffer from a serious medical condition, we strongly suggest that you consult with a licensed health care professional before using this product. Proper fitting is required for this product to be effective. Under some circumstances, this product may be prescribed by a physician. Please see the limited warranty for further information.

CARE INSTRUCTIONS
Hand wash only at 30° C. Wash with mild detergent; Air dry. Do not use bleach or other chemicals.

MATERIALS
Acetel, Polyester, ABS, Steel, Aluminum, UBL, EVA, Nylon, Silicone, Polyurethane.
NOT MADE WITH NATURAL RUBBER LATEX.

DISPOSAL
Product may be safely disposed in accordance with local laws.

REPORTING
Please report any serious incident involving the use of this device to both the Competent Authority in your state and to the manufacturer (listed on this document).

LIMITED WARRANTY
Aspen Medical Products, LLC, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products, LLC in the event of breach of warranty shall be to repair or replace the product or parts.

Aspen Medical Products, LLC shall have no obligation under this limited warranty in the event:
(a) The product was not purchased from Aspen Medical Products, LLC or through its authorized channels of distribution;
(b) The product is altered;
(c) Any parts not supplied by Aspen Medical Products, LLC are inserted into the product; or
(d) The product is not used in accordance with the Aspen Medical Products, LLC Instructions for Use.

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OR DEFECT IN THE PRODUCT. ASPEN MEDICAL PRODUCTS, LLC SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT, SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS, LLC.

This warranty gives you specific legal rights and you may have other rights which vary from location to location.
A listing of authorized distributors in your area is available upon request.

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Größe	Umfang
X-SMALL	26 – 31 in 66 – 78 cm
SMALL	30 – 37 in 76 – 94 cm
MEDIUM	36 – 43 in 91 – 109 cm
LARGE	42 – 49 in 107 – 124 cm
X-LARGE	48 – 55 in 122 – 140 cm
XX-LARGE	54 – 60 in 137 – 152 cm

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