# Getting started with GripAble Pro

# English

### Getting started with GripAble Pro

#### How to Start GripAble Pro

- 1. Charge Devices: Ensure both the GripAble Pro and the tablet are charged using the provided cables.
- 2. Turn On the Tablet: Press and hold the small button on the top left of the tablet for a few seconds to power it on.
- 3. Unlock the Screen: Swipe up from the bottom of the screen to unlock.
- 4. Log In: Enter the PIN 1234 and tap OK.
- 5. Open the GripAble App: Tap on the GripAble app icon.
- 6. Check Connectivity: Ensure the tablet is connected to the internet and Bluetooth is enabled. Swipe down from the top of the screen to access the settings icon in the bottom right and verify both are on/ activated.
- 7. Pair the Handset: Press the blue button at the top of the GripAble handset twice to connect it to the tablet via Bluetooth.
- 8. Return to App Home: Tap the three horizontal lines (III) on the bottom right of the tablet to return to the GripAble app's home screen.
- 9. Enter Code (if prompted): If asked, enter the 6-digit code you selected and press OK.
- 10. Select a Profile: Go to '**Profiles**' and choose one of the pre-made profiles (e.g., Eva 1 or Johanna X).
- 11. Start Training: Tap '**Train**' under the selected profile, then choose a game or activity and follow the instructions.

#### Account Management

(Each occupational therapist should have its own account.)

#### Switching Accounts:

1. Go to 'Settings' and select 'Sign out of Account'. Confirm with OK. 2. Tap on 'Create an account here,' then enter your email address and the 6-digit code.

#### Returning to the Original Account:

1. Go to 'Settings' and select 'Sign out of Account'. Confirm with OK.

2. Select 'Sign in with Email Address,' then enter your email and password.

#### Switching Off

• Turn Off the Tablet: Hold down the button at the top right of the tablet and select 'Power off'.

#### **Technical Support**

• Contact: For assistance, call GripAble UK support at +44 20 7661 4018 or email <u>hello@gripable.co.</u>

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